REF DualTIM

EN INSTRUCTIONS FOR USE

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INSTRUCTIONS FOR USE

Pedal exerciser, foldable

Product code: DualTIM

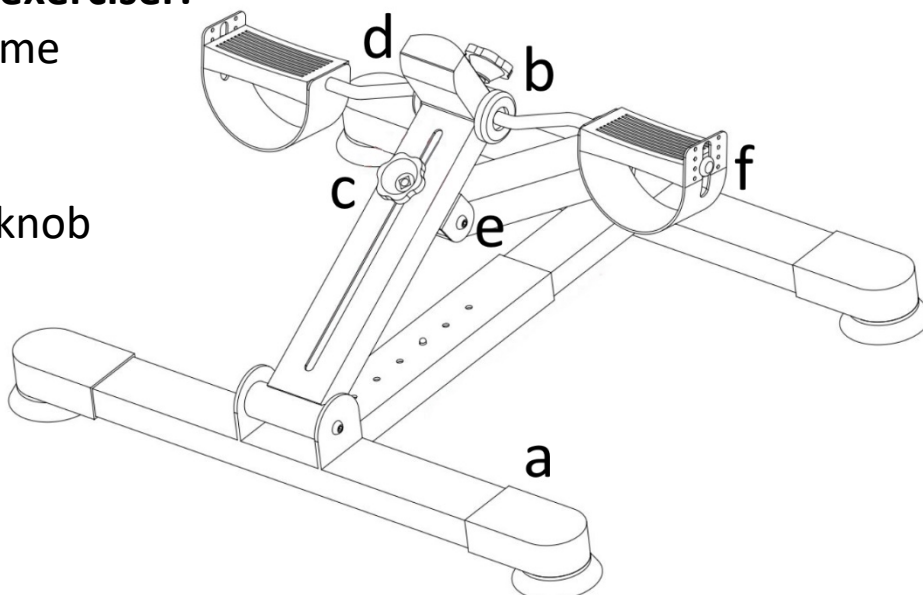
1. Characteristics

The lower limb pedal exerciser is made of a powder-coated steel frame. It is equipped with a pressure resistance mechanism, frame length adjustment, four anti-slip pads and rubber foot straps. The pedal exerciser has the option of changing from pedals to handles so it can be used for both lower and upper limb rehabilitation. The height adjustment feature allows for customisation to suit individual user needs. It has a rotary knob with a wide range of resistance adjustment, so it is also suitable for patients after extensive trauma. It can be used in hospitals, rehabilitation clinics and at home as a rehabilitation and personal recreation device.

Elements of the pedal exerciser:

- a. Pedal exerciser frame
- b. Adjustment knob
(shorter thread)
- c. Angle adjustment knob
with washer
(longer thread)
- d. Counter
- e. Pedal lock

2. Application

The pedal exerciser is designed for patients with contracture or muscular atrophy caused by long-term immobilisation due to a plaster bandage or surgery, as well as for patients who need to restore motor functions and re-learn the correct movement pattern due to damage to

the neurological system. It is possible to adjust the exerciser to the state of limb fitness and to the progress in rehabilitation owing to resistance adjustment.

3. Assembly and adjustment

- To assemble the rotor, join the two parts of the frame together and then twist them together using the angle adjustment knob (longer thread), placing the oblong washer beforehand.
- Insert the locking pin into the frame holes to secure the set angle of the pedal exerciser frame.
- Place the counter onto the mounting hole in the top of the frame.
- Screw the resistance adjustment knob (shorter thread) into the mounting hole in the top of the frame.
- To assemble the pedal exerciser, follow the above instructions in reverse order.
- To dismantle the counter or replace the battery in the counter, unscrew the adjustment knob completely and then dismantle the counter and/or replace the battery.
- To replace the battery, open the top cover of the counter by grasping the lower protruding part of the cover under the red MODE button.
ATTENTION: When replacing the battery, all data will be lost.

Height adjustment

The pedal exerciser is height-adjustable to suit the individual user's needs. There are 6 adjustment holes at the base of the pedal exerciser for height adjustment. The largest base spacing is the minimum height of the exerciser, the smallest spacing is the maximum height.

- Before changing the height adjustment, loosen the angle adjustment knob.
- Then press the locking pin and adjust the desired pedal exerciser base spacing. After adjustment, the locking pin should lock the selected setting.

- After changing the height adjustment, tighten the angle adjustment knob.

Change of pedals into handles / handles into pedals

The pedal exerciser is used for both lower and upper limb exercises (interchangeable). When changing the function of the pedal exerciser, it is necessary to change the pedals into handles (or handles into pedals) according to the following instructions:

- Unscrew the pedal lock (the locking handle allows it to be unscrewed without the need for additional tools).
- Replace pedal to handle (or handle to pedal).
- Tighten the pedal lock (the lock handle allows it to be unscrewed without the need for additional tools).
- Swap the other pedal/handle in a similar manner.

Resistance adjustment

In order to increase fitness levels and improve muscle work and muscle strength gains, the training intensity and/or duration of exercise should be gradually increased.

- To increase the resistance, turn the adjustment knob clockwise.
- To reduce the resistance, turn the adjustment knob anticlockwise.
- When performing low impact exercises, the adjustment knob should be completely unscrewed, whereas when performing resistance exercises, the adjustment knob should be tightened to the desired load.

Usage

- Place the pedal exerciser on the floor (lower limb exercises) or on a table (upper limb exercises). In order to prevent the pedal exerciser from moving, ensure that the anti-slip pads are firmly in place on the floor.

- Exercises should be carried out in a seated position on a stable surface. The pedal exerciser should be positioned exactly in front of the user.
- When exercising the lower limbs, the feet should be placed on the pedals inside the bands, which will prevent the foot from slipping off the pedal during exercise.
- For upper limb exercises, place your hands on the handles.
- Set the resistance using the adjustment knob.
- You can select the desired function on the pedal exerciser counter.

Counter function:

<p>CALORIES - <i>Calories</i> Counter of calories burnt</p>	
<p>TIMER - <i>Time</i> Duration of exercise</p>	
<p>DISTANCE - <i>Distance</i> Distance covered during exercise [km]</p>	
<p>CNT – <i>Counter</i> <i>Counter for exercise revolutions</i></p>	
<p>SCAN – <i>Function scan</i> Display of exercise parameters</p>	
<p>T.CNT – <i>Counter (total)</i> Total number of rotation of all exercises</p>	
<p>Auto Power ON - <i>Automatic start</i> Automatic start of the counter when activity is detected</p>	
<p>Auto Power OFF - <i>Automatic stop</i> Automatic stop of the meter after 5 min. of inactivity</p>	
<p>RESET - <i>Erase data</i> Press and hold the red button for min. 3 sec. (To erase T.CNT, remove the battery)</p>	

4. Technical data

Base dimensions	40 cm x 29-37 cm
Height of the pedal exerciser	22-30 cm
Maximum capacity	120 kg
Weight	5,30 kg
Battery	1 x 1.5V (LR44)
Frame colour	Black-red

5. Notes

Timago International Group does not bear responsibility of improper use of the pedal exerciser, failure to comply safety regulations and misuse.

6. Maintenance

Carry out regular maintenance of the technical condition of the pedal exerciser:

- It is recommended to clean the pedal exerciser regularly with a soft cloth and soapy water. After cleaning, dry the pedal exerciser thoroughly with soft cloth.
- Do not use strong polishing agents for cleaning as it may cause damage to the protective coatings.
- Remove impurities from moving elements of the exerciser on an ongoing basis.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the

product. Please, keep in mind that for warranty purposes, the proof purchase (receipt or invoice) should be retained.

9. Labels

	Reference numer		Manufacturer
	LOT numer		Manufacture date
	Serial numer		Medical device
	Note		Please read the manual
	The manufacturer checked for compliance with essential medical devices requirements.		

The entire range is available to you from our distributors.

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