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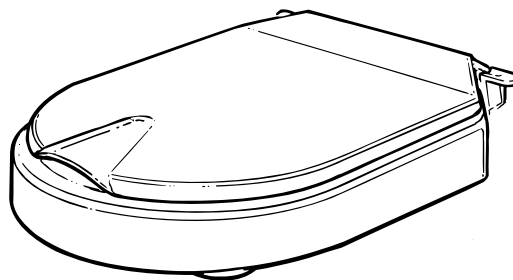
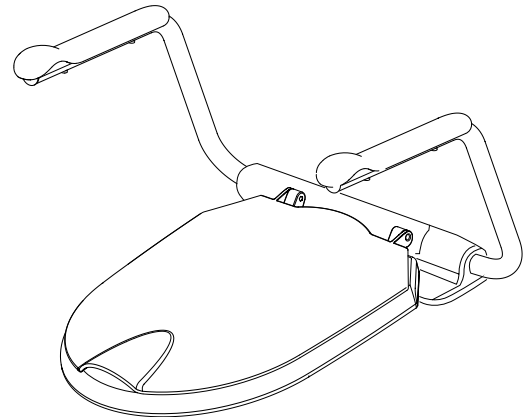
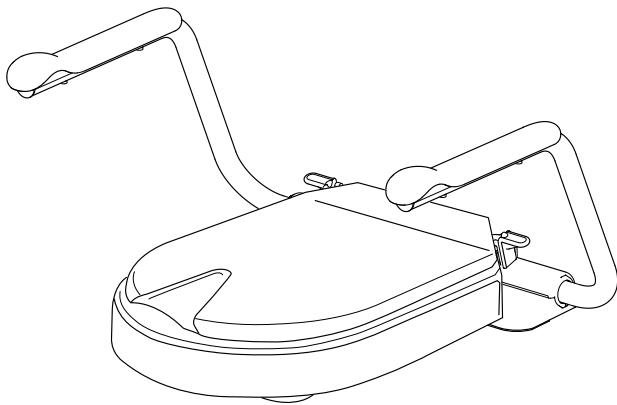
Hi-Loo/Supporter

**etac**<sup>®</sup>

Creating Possibilities

74949 - 2023-08-18 - Version 20

en pl

[www.etac.com](http://www.etac.com)

MD

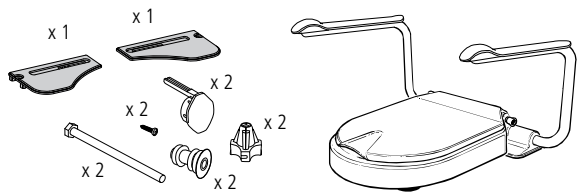
CE

en pl


A


1

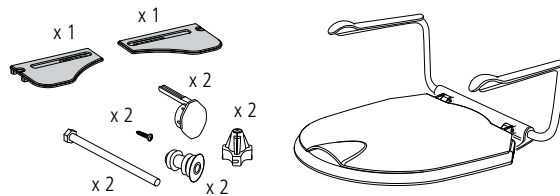
8030 1116-2 / 8030 1117-2 / 8030 1118-2 / \*8400 1291 / \*8400 1292


8030 3012-2 / \*8400 1293

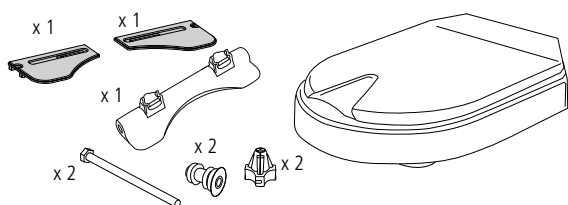
8030 1114-2 / 8030 1115-2

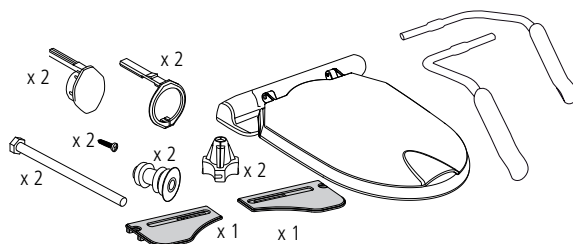
8030 3112-2 / \*8400 5101

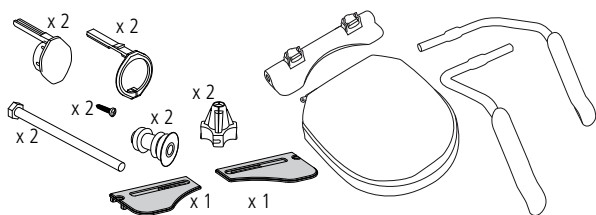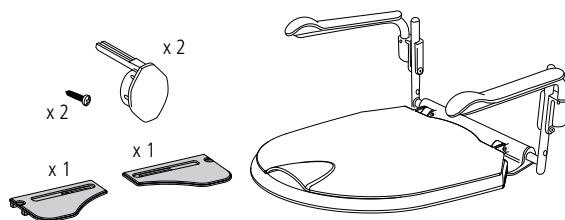
8030 1316-2 / 8030 1317-2 / 8030 1318-2 / \*8400 5099 / \*8400 5100


8030 3018-2 / \*8400 1294

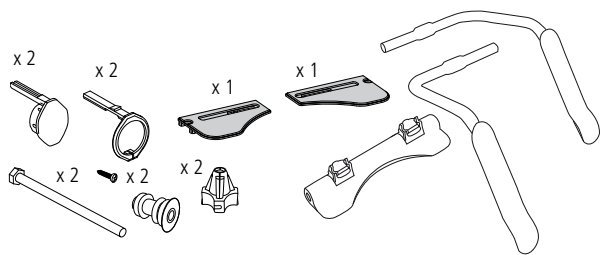8400 1299 / 8400 1322

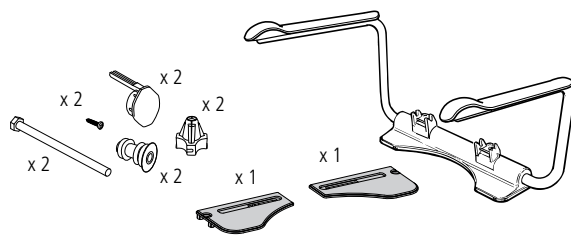8400 1297 / 8400 1298

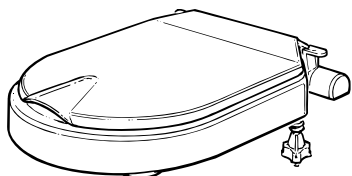8030 1106 / 8030 1107

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

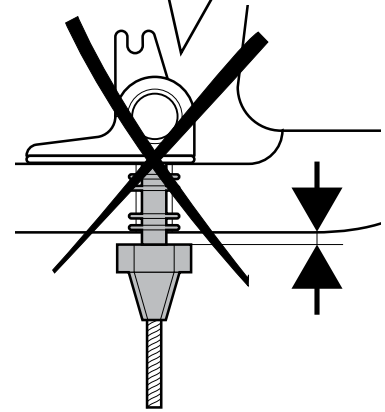
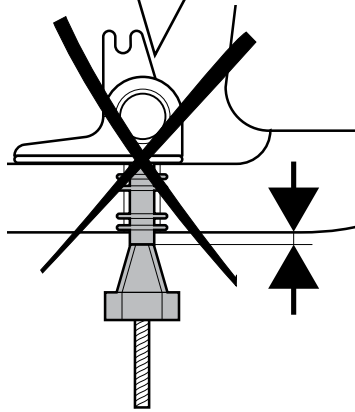
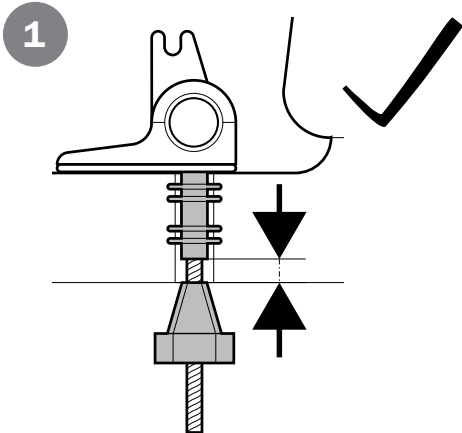
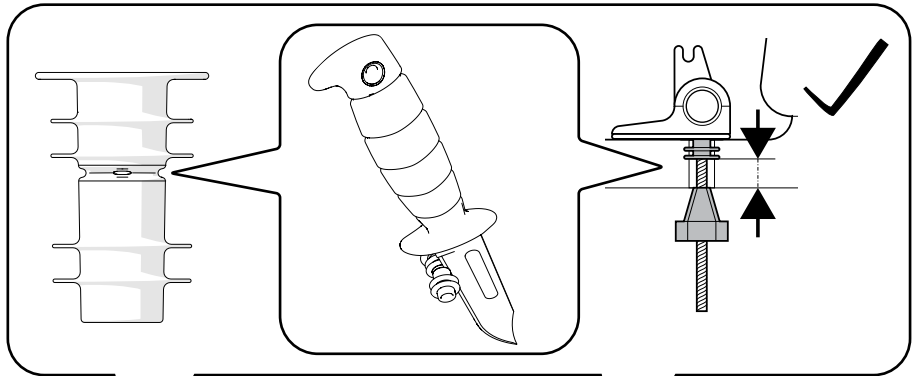
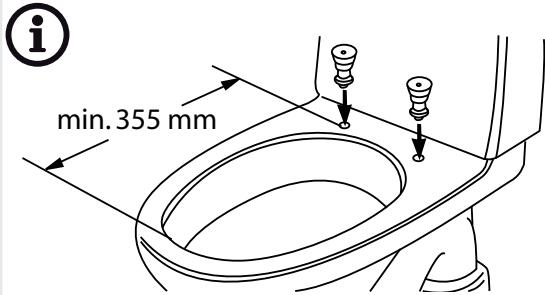
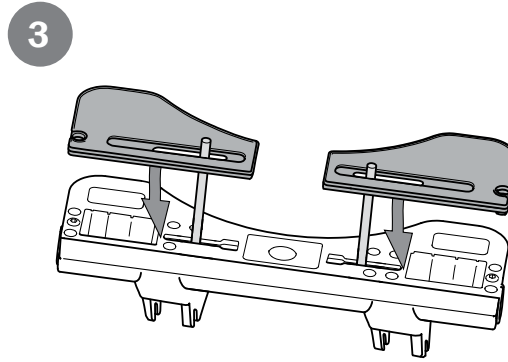
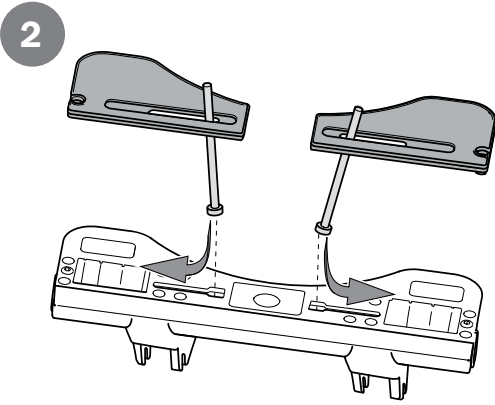
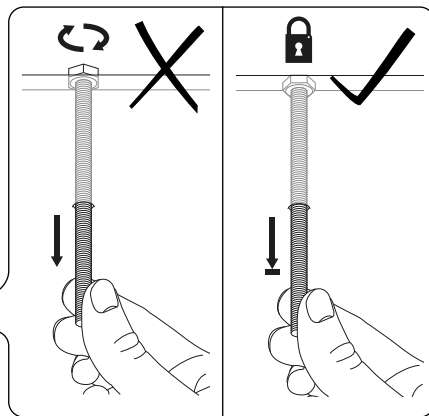
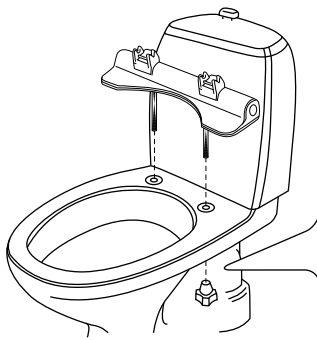
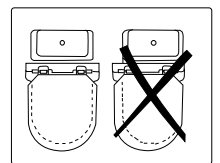
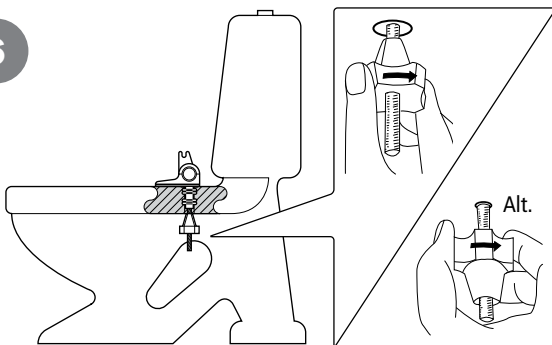
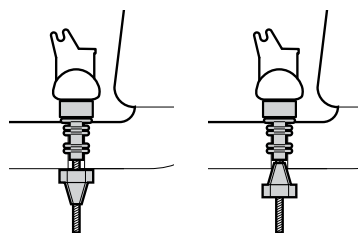
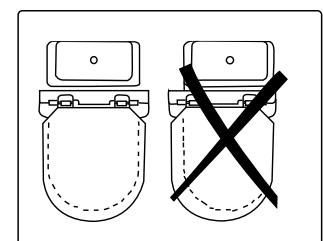
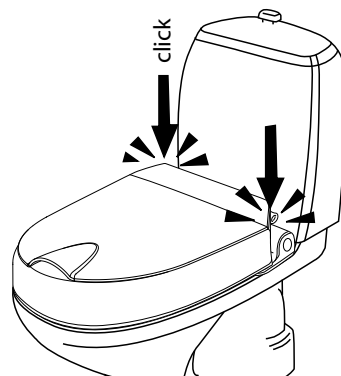
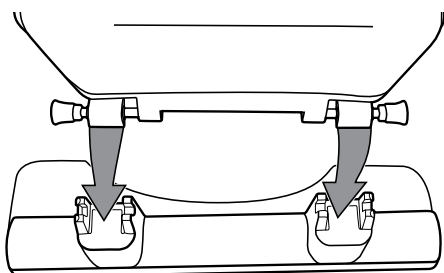
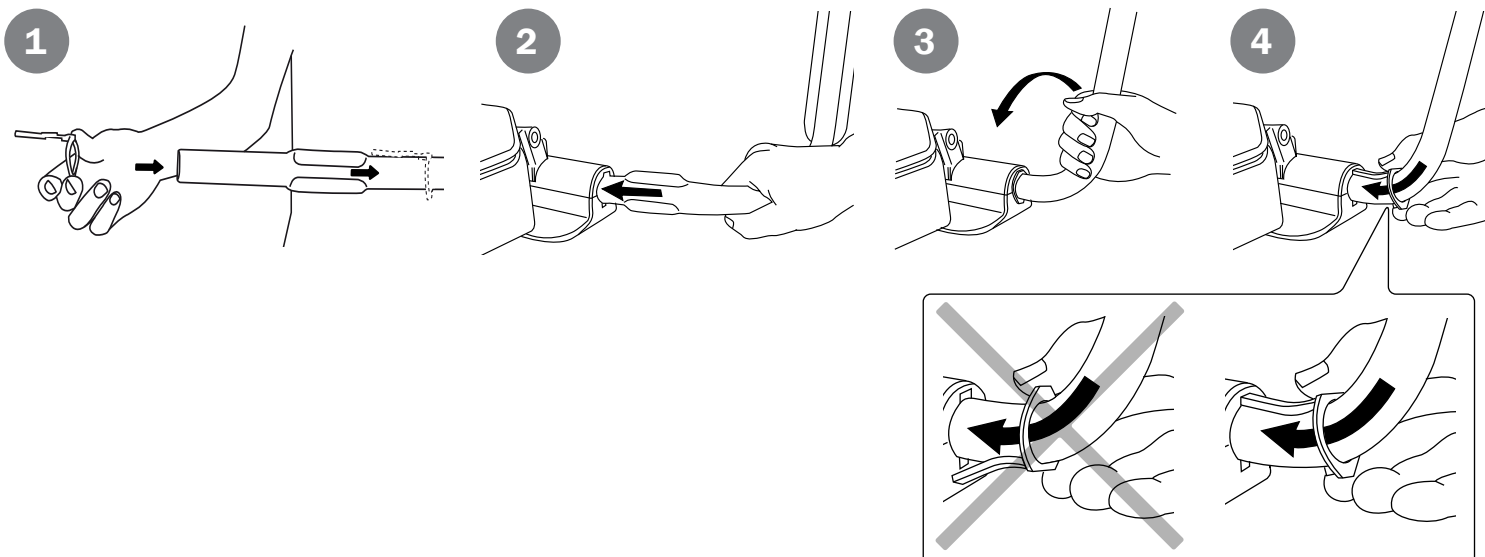
## G
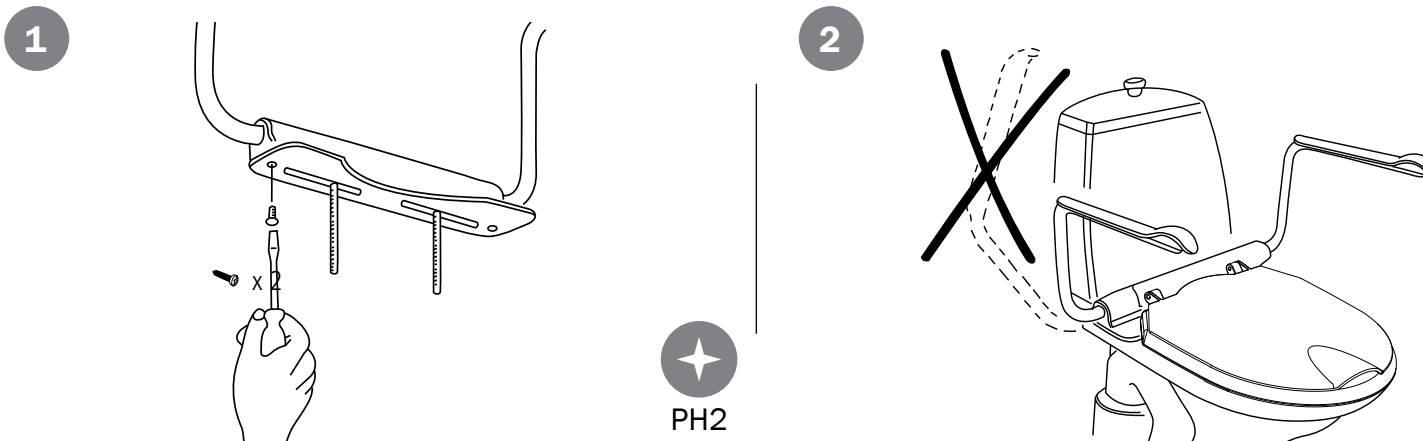
## H

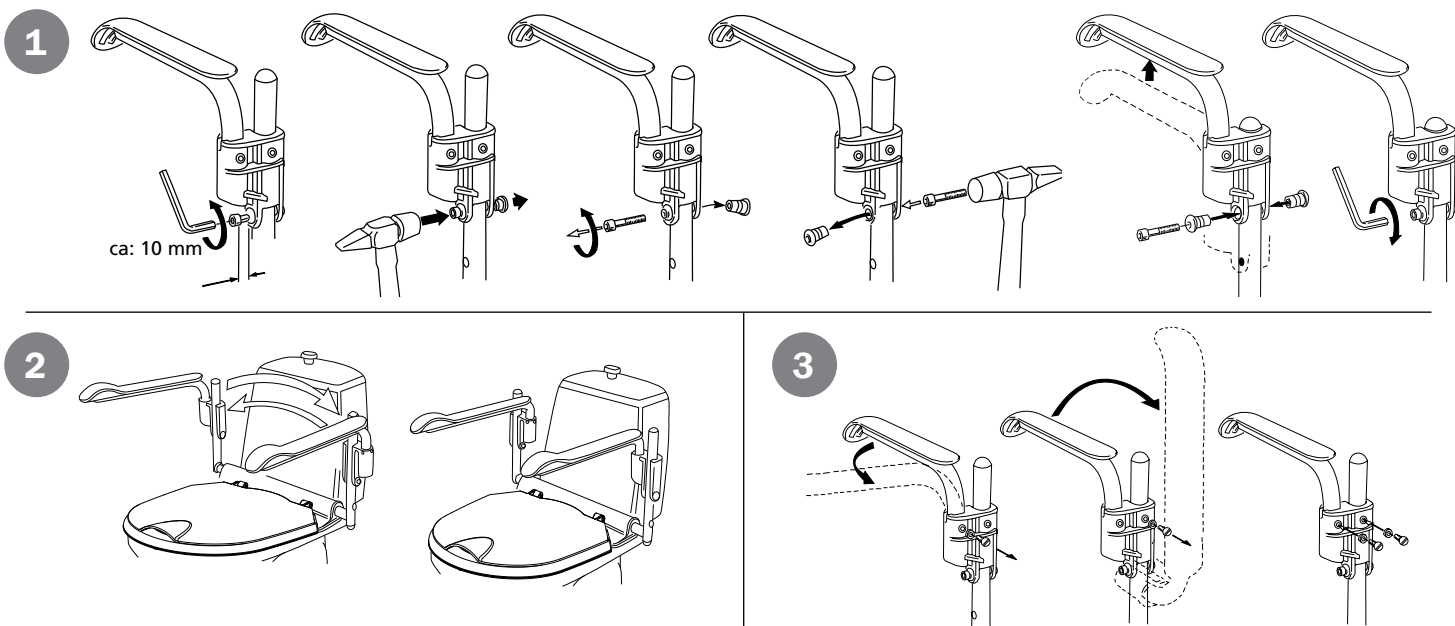Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

### Device description

Hi-Loo fixed and Supporter are raised toilet seats, available in different heights. They are fixed mounted on the toilet and can be used with one, two or without arm supports. The arm supports are individually foldable.

### Intended purpose

Hi-Loo fixed and Supporter (hereafter also referred to as "the device" or "the product"), are medical devices intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to facilitate access to the toilet for people with restricted functional ability. The raised seat and/or arm supports intend to provide stability and support when sitting down and standing up before and after performing toilet needs.

### Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more. Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

### Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

### Intended application

The device is intended for short-term and long-term use and can be applied several times per day. The device is intended to be used in contact with intact skin. The device is intended for refurbishment and reuse

### Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see [www.etac.com](http://www.etac.com).

### Indications

Indication for use is disability, including, but not limited to, an individual with fluctuating balance issues, physiological and/or functional impairment, and/or a history of falls.

### Contraindications

There are no known contraindications.

### Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

A defective device must not be used.

Head trap between toilet water cistern and the vertical part of the arm support

Do not stand on the device

### Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1, SS EN 12182.

### Pre-purchase information

Pre-purchase information can be found at [www.etac.com](http://www.etac.com).

### Adverse event

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

### Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

If there is any uncertainty regarding the execution of the customization, consult Etac. Device coupled with another device may not retain either device's CE mark unless combination agreements exist. Contact Etac for current information.

### Device description

**1.** Content of the delivery **2.** Seat raiser with lid **3.** Arm support **4.** Product label **5.** Item no label

..... [Figure A](#)

**Symbols** ..... [Figure B](#)

*Symbols in the manual and on the device:*

**1.** Warning, precaution or limitation. **2.** Useful advice and tips. **3.** Material for recycling. **4.** Tools screwdriver **5.** The commode must be securely anchored **6.** Maximum user weight (see technical data) **7-8.** Cleaning (see maintenance) **9.** Product weight

**Explanation item no label:** ..... [Figure C](#)

**1.** Product name **2.** Product description **3.** Serial number **4.** Item number **5.** Date of manufacture **6.** Barcode acc. to GS1-128 GTIN-14 and serial number\*

\*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

**Technical data** ..... [Figure D](#)

**Mounting** ..... [Figure E](#)

Toilet porcelain needs to be anchored to the floor securely

For wall hung porcelain, mount armrests before step 4.

**Mounting arm support** ..... [Figure F](#)

**Mounting fixed arm support** ..... [Figure G](#)

**Height adjustable arm supports** ..... [Figure H](#)

**1. Adjusting height 2. Adjusting width 3. Folding, fixing**

### Maintenance

Clean the device with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85 °C if so required.

Constituent materials are corrosion-resistant. 5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

### Storage

The device should be stored indoors in a dry place at a temperature above 5 °C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

### Accessories

- Adjustable arm supports for Supporter
- Seat unit for angled Hi-Loo
- Cross bar
- Screw and attachment kit special Supporter
- Attachment kit

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Heated device can cause burns

Ignoring installation or assembly instructions may result in injury.

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**Etac Ltd.**
**Promefa AG**
**Etac Supply Center AB**